**Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.**

**Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back.**

**Get the Blood Drop - Palm**

**ALTERNATIVE SITE TESTING (AST) PALM**

**Use the clear AST endcap.**

- Insert the lancet as described in *Prepare the Lancing Device*.
- Attach and lock the clear AST endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.
- “Click” to lock
- (If the device does not puncture, reset by repeating Step 4.)
- Attach and lock the endcap. Twist the white band to lock.

**CAUTION: Do not use the clear AST endcap on your fingertips.**

**5. Attach and lock the endcap. Twist the white band to lock.**

**6. Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.**

**7. Lift the lancing device straight up and away from the skin without smearing the blood drop.**

**8. Test immediately.**

**Cleaning**

**Clean your lancing device once a week. Use a cloth dampened with 70% isopropyl alcohol (found in pharmacies). Do not use bleach (sodium hypochlorite).**

**Supplies Needed for Cleaning:**

- A cloth dampened with 70% isopropyl alcohol (found in pharmacies).
- Do not use bleach (sodium hypochlorite).
- Paper towels

**Do not test the blood sample from palm if you get:**

- Smeared blood
- Runny blood
- Clotted blood
- Clear fluid mixed with the blood

**9. Wipe the puncture site with a clean, dry tissue and maintain pressure on the site until the bleeding stops.**

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**Customer Service**

**Warranty: Your lancing device is warranted for 5 years.**

**If you need assistance or have questions about blood sugar testing, please contact Customer Service at 0345 600 6030. See Customer Service Hours of Operation by visiting [www.diabetes.ascensia.com](http://www.diabetes.ascensia.com).**

**Ascensia, the Ascensia Diabetes Care logo, and Microlet are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.**

**All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.**

**© 2020 Ascensia Diabetes Care Holdings AG. All rights reserved. [www.diabetes.ascensia.com](http://www.diabetes.ascensia.com)**

**Ascensia Diabetes Care Holdings AG**

**Peter Merian-Strasse 90**

**4052 Basel, Switzerland**

**ASCENSIA Diabetes Care**

**For information on patents and related licenses, see [www.patents.ascensia.com](http://www.patents.ascensia.com) 90007626 Rev. 09/20**

**CE 2797**

**INSTRUCTIONAL INSERT**

- Lancet ejector for safe lancet disposal.
- Black/white adjustable endcap for fingertip testing only.
- Clear AST endcap for alternative site testing only.
- The MICROLET®NEXT lancing device and MICROLET® lancets are used to draw capillary blood from the fingertip or palm for blood glucose testing.

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## WARNING

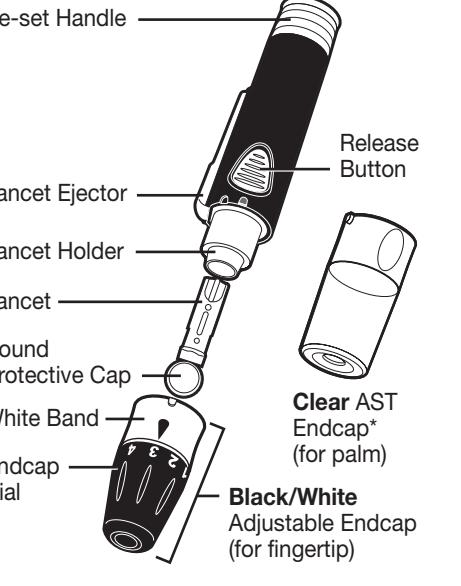
- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

### Do not use AST under the following conditions:

- If you think your blood sugar is low (hypoglycaemia).
- When blood sugar is changing rapidly (after a meal, insulin dose, or exercise).
- If you are unable to feel symptoms of low blood sugar (hypoglycaemic unawareness).
- If you get alternative site blood sugar results that do not agree with how you feel.
- During illness or times of stress.
- If you will be driving a car or operating machinery.

## POTENTIAL BIOHAZARD

- The MICROLET®NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.
- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- Always wash your hands with soap and water and dry them well before and after testing or handling the meter, lancing device, or test strips.

\* May not be included in all kits.  
Available on request from Customer Service.

## Prepare the Lancing Device

**Warning:** Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.

- Wash hands and the puncture site with soap and warm water. Rinse and dry well.
- Unlock the endcap. Twist the white band so the white raised mark and black indented mark line up as shown.
- Pull off the endcap.

"Click" to unlock

## Get the Blood Drop - Fingertip

**FINGERTIP TESTING**  
Use the black/white adjustable endcap. Do not use the clear AST endcap.

- Loosen the round protective cap on the lancet by rotating the cap  $\frac{1}{4}$  turn, but do not remove the cap.
- Insert the lancet firmly into the lancing device. Push until the lancet comes to a complete stop.
- Carefully twist off the round protective cap. Save the cap for disposing of the used lancet.

"Click" to lock

- Pull back the re-set handle to prepare the lancing device. The release button is raised when the lancing device is primed.

- Stroke your hand and finger toward the puncture site to form a drop of blood. Do not squeeze around the puncture site.
- Test immediately after you have formed a small, round blood drop, as shown.

- Press the lancing device firmly against the side of the finger and press the release button UNTIL THE DEVICE CLICKS.
- Rotate the endcap dial to adjust the puncture depth. The amount of pressure applied to the puncture site also affects puncture depth.

"Click"

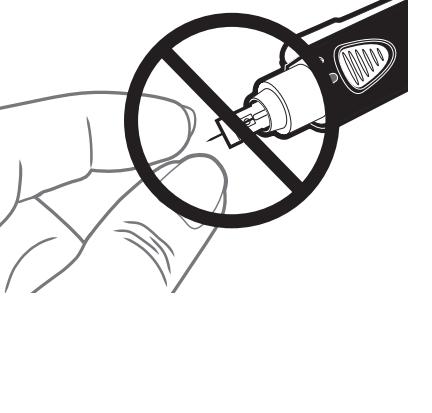
1 Shallowest Puncture      4 Deepest Puncture

- Wipe the puncture site with a clean, dry tissue and maintain pressure on the site until the bleeding stops.

## Eject / Dispose of the Lancet

Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.**

Your MICROLET NEXT lancing device has a lancet ejector. Do not use your fingers to remove the lancet from the lancing device.

- Place the round protective cap that you saved onto a flat surface. With the used lancet still in the lancing device, push the lancet into the center of the cap.

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